GB Instructions for use SAR TopHead
Contents:

1. Product description
2. Indication of the SAR TopHead
3. Contraindication of the SAR TopHead
4. Disinfection and cleaning
5. Recommended uses

1. Product description:

Dear Customers

Please read these instructions for use very carefully before using the SAR TopHead and instruments. The SAR TopHead was developed for use by qualified dentists and should only be used by them. The user is obliged to accept responsibility for checking that the material is suitable for the intended use. This is particularly required if it involves use that is not listed in the product information. The manufacturer does not accept liability for damage caused by failure to observe the guidelines provided by the instructions for use or use that is not compatible with the system.

2. Indication of the SAR TopHead:

SAR TopHeads have been developed for repairing fractured root posts fabricated using any alloy.

3. Contraindication of the SAR TopHead:

SAR TopHeads should never be used for other than the methods listed below. The stated speeds must be observed.

4. Disinfection and cleaning:

The SAR head-cutting diamond should be disinfected, sterilised in an autoclave (134°C / 273° F in the usual way and reused.

RDG thermal disinfection can also be used. In Germany use only a VAH (German Association for Applied Hygiene) approved disinfectant solution (without alcohol). The glass-fibre SAR TopHead heads can be disinfected and cleaned of traces of grease using medical alcohol. Sterilisation is not possible. Do not touch with bare fingers after cleaning. The SAR TopHead heads are single-use products. They may not be reused to avoid the risk of contamination. Adhere to the Robert-Koch Institute guidelines (hygiene requirements in dentistry) and the valid legal and hygiene regulations in your region!

Detailed preparation recommendations according to DIN EN ISO 17664

➔ www.nti.de

5. Recommended uses:

The SAR head-cutting diamonds should be used with a blue contra-angle.

Working speeds: 1,000 - 1,500 rpm.

An X-ray should be taken and the integrity of the tooth checked before the repair. The entire working procedure should be performed with the use of a rubber dam.

1. Grind the tooth surface and defect post smooth using a diamond instrument.
2. Trim the seat for the TopHead around the fractured post using the SAR head-cutting diamond (hollow inside). Ensure that the head-cutting diamond is moved along a single axis around the fractured post.
3. Use only water to rinse the tooth. The use of sodium hypochloride and hydrogen peroxide is contraindicated, as both substances negatively influence the polymerisation of the bonder and adhesive.
4. Condition the surface of the tooth and the seat of the SAR TopHead head in the total etch (full etch), etch and rinse technique.
5. Apply primer and dual-curing bonder according to the bonder and adhesive system manufacturer's instructions.
6. Remove the SAR TopHead head from the packaging using tweezers, coat the inside and outside with dual-curing composite and insert over the post into the seat. Slightly tap with an instrument or use an ultrasonic ball to ensure a gap-free fit.
7. At the same time, build up a core using the dual-curing composite used for adhesive retention of the head. This creates a monoblock between the adhesive and core, which increases the stability.

The Voco (Futurabond and Rebuilda DC) and DMG (Luxabond DC and Luxacore DC) systems are recommended for adhesive retention.

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