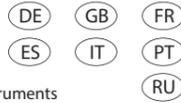
NTI-Kahla GmbH • Rotary Dental Instruments  
Im Camisch 3 • D-07768 Kahla/Germany  
Tel. +49(0)36 424-573-0 • Fax +49(0)36 424-573-29  
e-mail: nti@nti.de • www.nti.de

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## GB Instructions for use for dentistry bur blocks

### Contents:

#### 1. Product description

#### 2. Indication for bur blocks

### 3. Contraindication for bur blocks

#### 4. Notes on processing

##### 1. Product description

Dear customer,

The bur blocks with the order numbers 4056, 4057, 4061 and the thermal disinfection bur block 4066 were developed for use in dentistry.

Please read the instructions for use carefully before using the bur blocks. The bur blocks were developed for use by trained and experienced dentists and professionals and may only be used by these.

The users are obliged to check the material for its suitability for the intended use at their own responsibility. This is particularly necessary if this concerns a use not mentioned in the instructions for use.

The bur blocks No. 4056, 4057 and 4061 are made of aluminium (base) and stainless steel (lid).

The thermal disinfection bur block (4066) is made of non-corrosive stainless steel.

The manufacturer shall not be liable for any damages due to non-compliance with these provisions in the instructions for use or which are due to system-incompatible use.

##### 2. Indication for bur blocks

The bur blocks were developed for the following areas of use:

###### 2.1. 4056:

The bur block was developed to hold FG instruments. Processing in the thermal disinfection bur block is only possible without instruments. Cleaning of instruments is performed with the "Thermal disinfection bur block" 4066. Manual disinfection and sterilisation can be performed with instruments.

###### 2.2. 4057:

The bur block was developed to hold FG and RA (right angle) instruments. Processing in the thermal disinfection bur block is only possible without instruments. Cleaning of instruments is performed with the "Thermal disinfection bur block" 4066. Manual disinfection and sterilisation can be performed with instruments.

###### 2.3. 4061:

The bur block was developed specifically to hold RA instruments (right angle) with a long shank or working part. Processing in the thermal disinfection bur block is only possible without instruments. Cleaning of instruments is performed with the "Thermal disinfection bur block" 4066. Manual disinfection and sterilisation can be performed with instruments.

###### 2.4. 4066 – "Thermal disinfection bur block":

The bur block was developed specifically for cleaning all rotating instruments (FG/RA/HP) in the thermal disinfection bur block. Manual disinfection and sterilisation can be performed with instruments. Only use without lid in the WD!

##### 3. Contraindication for bur blocks

The bur blocks may under no circumstances be used outside the areas listed under Section 2. The bur blocks (4056, 4057, 4061) may under no circumstances be placed in the thermal disinfection bur block with instruments!

### 4. Notes on processing

The bur blocks are supplied non-sterile. Sterilisation is determined by the practice and specific use hygiene plan in the dental practice.

Clean, check, disinfect and sterilise the blocks as follows: use an approved aldehyde-free disinfectant VAH (D) or FDA (US) or with CE (EU) certificate.

**Attention:** Never use alcohol-based disinfectants! Possibly with additional cleaning in an ultrasonic bath, observe the concentrations according to the instructions of the manufacturer of the disinfection bath.

The 4066 bur block for thermal disinfection was developed specifically for cleaning rotating instruments in the WD (washer and disinfection device). Insert the instruments into the corresponding silicone holders. The larger holders are intended for HP and RA instruments and the smaller holders for FG instruments. Remove the block from the WD immediately after completion of the programme and dry with air. Do not mount the lid in the thermal disinfection!

The bur blocks (No. 4056, 4057 and 4061) can be cleaned in a thermal disinfection. This must always be performed without instruments! Open the empty block at an angle of 90° and place upside down on the rack of WD. This is the only way to guarantee that the holes are rinsed and that the water can drain afterwards.

Checking the bur blocks: it is essential to check the holes after removal from the WD and to remove any liquids with oil-free air. The blocks must not remain in the WD overnight. This avoids stains and discolouration.

Then load with instruments, seal as required, and sterilise in an autoclave (134°C / 273°F; 2.1 bar; 5 min) and reuse.

**Detailed recommendations for cleaning and care in accordance with DIN EN ISO 17664** [www.nti.de](http://www.nti.de)

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© NTI-Kahla GmbH Rotary Dental Instruments